

Chin Rest(Head and Chin)

This high quality chin rest is frequently used in medical, visual field and other medical or ophthalmic applications.



When To Use

Intended for visual field related training. Light weight, small, and easily transported the chin rest is suitable to be used in the community at client's homes, in therapy rooms or offices. Many of our chin rests are also used in medical trials, photography and studies that require the support of the head.

How To Use

When required, simply fixes to the edge of any table with the screw clamp.

The total height and the forehead support can be modified in seconds to suit the user. There is a comfortable notched support for the chin.

The head band can be detached if necessary.

How To Clean

Wipe down with antibacterial cloth / spray to disinfect

Specification

Material: White enamel alloy

Weight: 1.10kg

Desk to Chin: Min 265mm to Max 375mm

Chin to Midpoint Forehead Rest: Min 30mm to Max 165mm

Total Height Table to Forehead: Min 295mm to Max 540mm

Max table thickness for clamp: 45mm

